# Protocols for Laboratory Verification of Performance of the FilmArray<sup>®</sup> Respiratory Panel (RP)

### **Purpose**

The Clinical Laboratory Improvement Amendments (CLIA), passed in 1988, establishes quality standards for all laboratory testing to ensure the accuracy and reliability of patient test results, regardless of where the test is performed. The CLIA regulations include a requirement for verifying the performance specifications of unmodified, moderate complexity tests cleared or approved by the FDA.

This document provides examples of procedures to assist your laboratory in developing a protocol for the verification of FilmArray RP performance on FilmArray<sup>®</sup> 1.5, 2.0 and Torch Systems as required by CLIA. Two possible verification schemes, compatible with the FilmArray RP, have been designed. Each verification scheme provides positive and negative tests for each organism detected by the FilmArray RP using non-clinical specimens and may be easily modified or expanded to meet specific criteria. Day-to-day variation is evaluated by testing each sample on two separate days. To evaluate user-to-user variation, multiple laboratory technicians may test the same sample. In addition, testing patient samples for verification or to evaluate matrix effects on the performance of the FilmArray RP should be done under the guidance of the Laboratory Director, but is not described here.

As per the CLIA regulation, the Laboratory Director is ultimately responsible for ensuring that verification procedures meet the appropriate standards for CLIA and applicable laboratory accrediting agencies.

## FilmArray Intended Use

The FilmArray RP is a multiplexed nucleic acid test intended for use with FilmArray Systems for the simultaneous qualitative detection and identification of multiple respiratory viral and bacterial nucleic acids in nasopharyngeal swabs (NPS) obtained from individuals suspected of respiratory tract infections. The following organisms and subtypes are identified using the FilmArray RP: adenovirus, coronavirus 229E, coronavirus HKU1, coronavirus NL63, coronavirus OC43, human metapneumovirus, influenza A, influenza A subtype H1, influenza A subtype H3, influenza A subtype 2009 H1, influenza B, parainfluenza virus 1, parainfluenza virus 2, parainfluenza virus 3, parainfluenza virus 4, rhinovirus/enterovirus, respiratory syncytial virus, *Bordetella pertussis, Chlamydia pneumoniae*, and *Mycoplasma pneumoniae*.

The complete intended use statement and additional information about the use of the FilmArray System can be found in the *FilmArray Respiratory Panel Instruction Booklet*.

**Note:** Chlamydia pneumoniae is sometimes referred to as Chlamydophila pneumoniae.

## TECHNICAL ::: NOTE

## **Performance Verification: Overview**

Each procedure described below will generate multiple positive and negative results for each of the FilmArray<sup>®</sup> Respiratory Panel (RP) assays. The procedures were developed using a Respiratory Verification Panel available from ZeptoMetrix Corporation, Buffalo, NY (part number NATRVP-IDI).

Two different examples of performance verification procedures are described: (1) a Simple Protocol for the verification of the FilmArray RP performance on any FilmArray<sup>®</sup> System and (2) a Viral Transport Media (VTM) Protocol that evaluates FilmArray RP performance when organisms are in a VTM sample matrix.

A FilmArray System is defined as one or more FilmArray<sup>®</sup> Instruments or modules connected to and controlled by a single computer system. If the laboratory director chooses not to perform the verification protocol on each individual instrument, it is advised that test replicates are evenly distributed among the instruments or modules.

Performance verification protocols should be designed to take advantage of the multiplex nature of the FilmArray RP. Verification testing efficiency is maximized by evaluating multiple target organisms in a single test run.

Clinical/patient specimens may be used in place of or in addition to the verification protocol examples described here in order to assess clinical sensitivity and sample matrix effects in the performance verification of the FilmArray RP.

Verification Protocol	Organisms per Pool <sup>a</sup>	Number of Sample Pools	Replicates per Sample Pool	Pouches Required	Expected Positive Results	Expected Negative Results	Approximate Days of Testing <sup>b</sup>
Example 1: Simple protocol	2 or 3	7	4	28	4 per organism	24 per organism	6
Example 2: Viral Transport Medium (VTM) protocol	1	19	4	76	4 per organism	72 per organism	14

#### Table 1. Overview of Verification Protocols

<sup>a</sup> Depending on the material used for verification, pooling of organisms may not be appropriate and the values in the table may need to be modified.

<sup>b</sup> The approximate number of days for testing assumes a system configured with one instrument.

## **Performance Verification: Materials**

The following materials may be used to perform verification procedures:

#### Table 2. Materials needed for recommended verification protocols

Material	Part Number			
FilmArray RP Kit (30 tests)	BioFire Diagnostics, LLC RFIT-ASY-0124			
Control Organism	ZeptoMetrix NATRVP-IDI <sup>a</sup>			
Transport Medium (e.g. Remel M4 Viral Transport Media)	Various media are appropriate			

<sup>a</sup> Any appropriate source of organism may be used for verification of any or all of the assays in the FilmArray RP. However, when alternate organism sources are used (i.e. not the ZeptoMetrix NATRVP-IDI material), the sample volumes or pooling schemes suggested in the examples below may need to be adjusted.

# TECHNICAL ::: NOTE

### Simple Protocol

The Simple protocol utilizes samples prepared by pooling together either 2 or 3 different organisms (ZeptoMetrix NATRVP-IDI control organism). The proposed organism pooling scheme (Table 3) should be followed to obtain the expected number of positive and negative results for each assay in a time and resource-efficient manner.

**Note:** Dilution of ZeptoMetrix Respiratory Verification Panel organisms beyond levels proposed in these guidelines may lead to inconsistent results and is not recommended.

The Simple Protocol can be followed to test a total of 28 pooled samples, providing 4 positive results and 24 negative results per assay. The number of samples tested per day should be determined by the individual laboratory. The testing scheme can be modified to run more samples per day based on the number of instruments configured on the FilmArray<sup>®</sup> System.

Pooled samples can be stored overnight (or up to 3 days) at refrigeration temperature (2–8°C) for subsequent testing to evaluate day-to-day variation.

Organism	Approximate volume	Approximate Final Volume of Pool		
Pool 1				
Adenovirus	0.6 mL	1.2 mL		
Influenza A subtype H1	0.6 mL	1.2 IIIL		
Pool 2				
Influenza B	0.6 mL			
Parainfluenza virus 4	0.6 mL	1.8 mL		
Coronavirus OC43	0.6 mL			
Pool 3				
Human rhinovirus/Enterovirus	0.6 mL			
Influenza A subtype H3	0.6 mL	1.8 mL		
Coronavirus 229E	0.6 mL			
Pool 4				
Parainfluenza virus 1	0.6 mL			
Parainfluenza virus 2	0.6 mL	1.8 mL		
Mycoplasma pneumoniae	0.6 mL			
Pool 5				
Influenza A subtype H1-2009	0.6 mL	1.2 mL		
Parainfluenza virus 3	0.6 mL			
Pool 6				
Respiratory Syncytial Virus	0.6 mL			
Coronavirus NL63	0.6 mL	1.8 mL		
Human Metapneumovirus	0.6 mL			
Pool 7				
Bordetella pertussis	0.6 mL	1.8 mL		
Chlamydia pneumoniae	0.6 mL			
Coronavirus HKU1	0.6 mL			

#### Table 3. Proposed Organism Pooling Scheme

An example Simple Protocol workflow is provided below.

# TECHNICAL ::: NOTE

### **Simple Protocol Example**

The estimated total time for completion for this Simple Protocol verification example is 6 days for a FilmArray<sup>®</sup> System configured with 1 instrument. A proposed organism pooling scheme is presented above in Table 3.

### <u>Day 1</u>

1. Prepare three sample pools (i.e. pools #1, #2, and #3 in Table 3 above) from ZeptoMetrix NATRVP-IDI control material.

**Note:** It is important to prepare only the number of sample pools that will be tested within 3 days of preparation. The suggestion to prepare 3 sample pools is based on testing 6 pouches per day. The number of samples prepared may be increased or decreased based on the laboratory's work schedule and number of instruments connected within a FilmArray System.

- a. Use a transfer pipette to remove the entire contents of the ZeptoMetrix organism vial (approximately 0.6 mL) and transfer to a new vial or tube.
- b. Repeat with the second (and third) organism to combine the appropriate organisms into a single vial or tube (approximately 1.2 mL total volume for two organisms or 1.8 mL for three organisms).

**Note:** Pools 1 and 5 contain 1.2 mL and will be used to test four pouches (approximately 0.3 mL for each pouch). If the volume of the sample is not sufficient to reach the 3<sup>rd</sup> line of the Transfer Pipette when loading sample into your pouch, note the volume shortage and proceed with testing.

- c. Ensure the pooled sample is effectively mixed by vortexing prior to testing.
- 2. Test two samples from a single sample pool. The duplicate samples should be tested in a single day by different users.

**Note:** For each sample, follow instructions in the *FilmArray*<sup>®</sup> *Respiratory Panel Instruction Booklet* and *FilmArray Respiratory Panel Quick Guide* for pouch preparation, pouch hydration, sample loading, and sample testing.

- 3. Repeat Step 2 for the remaining sample pools (i.e. pools #2 and #3) to be tested that day.
- 4. Refrigerate samples (2–8°C) for up to 3 days for the evaluation of day-to-day variation.

#### <u>Day 2</u>

To evaluate day-to-day variation, test the remaining volume of the sample pools prepared on Day 1 by repeating Step 2 above.

### <u>Day 3</u>

Prepare 3 new sample pools (i.e. pools #4, #5, and #6) as described in Step 1. Test samples according to Step 2.


#### <u>Day 4</u>


To evaluate day-to-day variation, test the samples prepared on Day 3 by repeating Step 2.

## TECHNICAL ::: NOTE

## <u>Day 5-6</u>

Repeat Steps 1–2 as above for any remaining sample pools (e.g. pool #7).

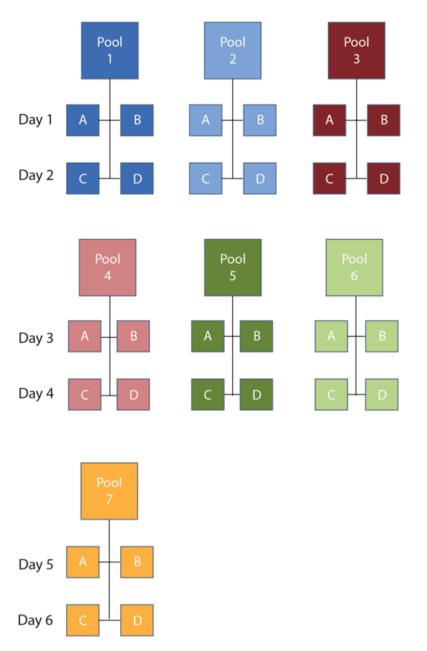

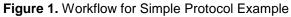

## TECHNICAL ::: NOTE

## **VTM Protocol**

The VTM Protocol utilizes samples prepared by adding a single control organism (ZeptoMetrix NATRVP-IDI control organism) to an equal volume of Viral Transport Medium (VTM) sample matrix.

The VTM Protocol can be followed to test a total of 76 samples, providing 4 positive results and 72 negative results per assay. The number of samples tested per day should be determined by the individual laboratory. The testing scheme can be modified to run more samples per day based on the number of instruments or modules configured on the FilmArray<sup>®</sup> System.

VTM samples can be stored overnight (or up to 3 days) at refrigeration temperature (2–8°C) for subsequent testing to evaluate day-to-day variation. To evaluate user-to-user variation, multiple laboratory technicians may perform testing.

Table 4.	Proposed	VTM Sam	ple Preparati	on Scheme
----------	----------	---------	---------------	-----------

VTM Sample				
	Approximate volume			
ZeptoMetrix NATRVP-IDI Control Organism	0.6 mL			
VTM	0.6 mL			

An example VTM Protocol workflow is provided below.

### **VTM Protocol Example**

The estimated total time to completion for this VTM Protocol verification example is 14 days for a FilmArray System configured with 1 instrument.

### <u>Day 1</u>

1. Prepare three VTM Samples (i.e. sample #1, #2, #3) by mixing ZeptoMetrix NATRVP-IDI control material with VTM as proposed in Table 4.

**Note:** It is important to prepare only the number of samples that will be tested within 3 days of preparation. The suggestion to prepare 3 samples is based on testing 6 pouches per day. The number of samples prepared may be increased or decreased based on the laboratory's work schedule and number of instruments configured on the FilmArray System.

- a. Pipet 0.6 mL of VTM into a sterile tube or vial.
- b. Use a transfer pipette to remove the entire contents of the ZeptoMetrix organism vial (approximately 0.6 mL) and add organism to the tube or vial containing VTM (total sample volume for one organism in VTM will be approximately 1.2 mL).
- c. Ensure the VTM sample is effectively mixed by vortexing prior to testing.
- 2. Test two samples (pouches) from a single VTM sample. The duplicate samples should be tested in a single day by different users.

**Note:** For each sample, follow instructions in the *FilmArray*<sup>®</sup> *Respiratory Panel Instruction Booklet* and *FilmArray Respiratory Panel Quick Guide* for pouch preparation, pouch hydration, sample loading, and sample testing.

# TECHNICAL ::: NOTE

3. Repeat Step 2 for the remaining samples to be tested that day.

**Note:** The approximate volume of VTM samples is 1.2 mL, which should be sufficient to test four pouches per sample (approximately 0.3 mL for each pouch). If the volume of the sample is not sufficient to reach the 3<sup>rd</sup> line of the Transfer Pipette when loading sample into your pouch, note the volume shortage and proceed with testing.

4. Refrigerate samples (2–8°C) for up to 3 days for the evaluation of day-to-day variation.

### <u>Day 2</u>

To examine day-to-day variation, test the samples prepared on Day 1 by repeating Step 2.

### <u>Day 3-4</u>

Repeat Steps 1–4 for 3 additional samples (i.e. #4, #5, and #6).

#### <u>Day 5–6</u>

Repeat Steps 1–4 for 3 additional samples (i.e. #7, #8, and #9).

#### <u>Day 7–8</u>

Repeat Steps 1–4 for 2 additional samples (i.e. #10, #11, and #12).

#### Day 9-10

Repeat steps 1–4 for the remaining samples (i.e. #13, #14 and #15).


#### Day 11-12

Repeat steps 1–4 for the remaining samples (i.e. #16, #17, and #18).

### Day 13-14

Repeat steps 1–4 for the remaining samples (i.e. #19).

## TECHNICAL ::: NOTE

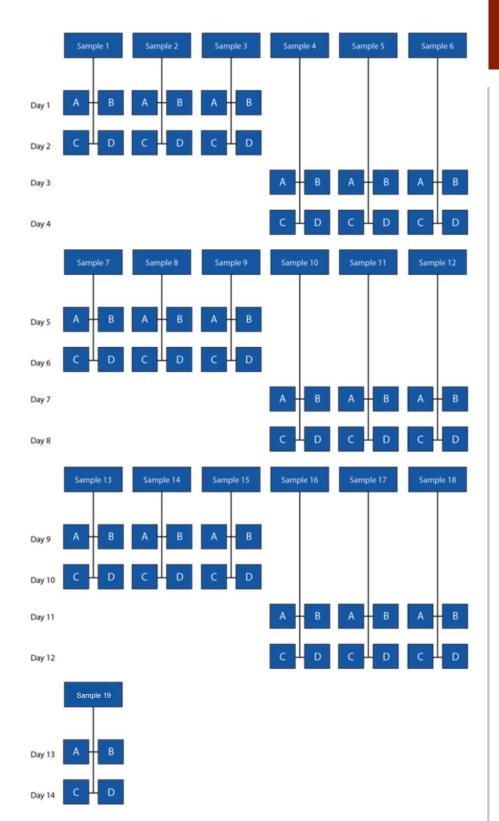

# TECHNICAL ::: NOTE

Figure 2. Workflow for VTM Protocol Example

## Verification of loaner and repaired instruments

If it becomes necessary to verify the performance of a loaner or repaired instrument, the following protocol <u>may</u> serve as a guideline, but should be verified by the Laboratory Director.

- Select a few specimens and/or proficiency samples (any combination of positives and negatives) previously tested on the FilmArray<sup>®</sup> Respiratory Panel (RP). The Laboratory Director should determine the appropriate number of samples to test. Two to six samples may be sufficient. Proficiency samples should not be pooled or diluted.
- 2. Test the selected specimens/samples on the loaner or repaired instrument and document the results.

## **Technical Support Contact Information**

BioFire is dedicated to providing the best customer support available. If you have any questions or concerns about this process, please contact the FilmArray Technical Support team for assistance.

**BioFire Technical Support** Email: <u>support@biofiredx.com</u> Phone: +1-801-736-6354, select Option 5



A BIOMÉRIEUX COMPANY

# TECHNICAL ::: NOTE

## FilmArray<sup>®</sup> Respiratory Panel (RP) Verification Record

Computer System Serial Number #: \_\_\_\_\_

FilmArray RP Kit Part #: \_\_\_\_\_ Lot #: \_\_\_\_\_

Organism/Sample Source and Lot #: \_\_\_\_\_

## TECHNICAL ::: NOTE

Organism	Instrument Serial #	Was the Organism Detected?	No. Positive	No. Negative	No. Days Tested	No. Users	Patient Samples Tested?
Adenovirus		Yes No					
Bordetella pertussis		Yes No					
Chlamydia pneumoniae		Yes					
Coronavirus 229E		Yes					
Coronavirus NL63		Yes No					
Coronavirus OC43		Yes No					
Coronavirus HKU1		Yes No					
Human Metapneumovirus		Yes No					
Human Rhinovirus/Enterovirus		Yes No					
Influenza A subtype H1		Yes					
Influenza A subtype H1-2009		Yes					
Influenza A subtype H3		Yes					
Influenza B		Yes					
Mycoplasma pneumoniae		Yes					
Parainfluenza virus 1		Yes					
Parainfluenza virus 2		Yes					
Parainfluenza virus 3		Yes					
Parainfluenza virus 4		Yes					
Respiratory Syncytial Virus		Yes No					

Reviewed by:

Signature

Date